

RADIOLUCENT WRIST FIXATOR

Please kindly refer to the product IFU PQWRI, to the Orthofix implantable devices and related instrument IFU PQSCR, and to the reusable medical devices IFU PQRMD that contain instructions for use of the product.

INSERTION OF METACARPAL SCREWS

- Apply the fixator either laterally in the frontal plane or at an angle of about 45° from the frontal plane. Use a tourniquet. Insert the proximal metacarpal screw first. Make a longitudinal stab incision close to the base of the bone, on the flare of the tubercle of the second metacarpal, and dissect the soft tissues down to the bone. Remove the threaded guide from the template and position the end of the fixed guide close to the base of the bone, on the flare of the tubercle and in the central axis. Insert a drill guide into the fixed guide and drill the bone with a 2.7mm drill bit.



WARNING: Do not excessively penetrate the second cortex with any type of screws to avoid soft tissue damage.

- Remove the drill guide and insert a 70/20mm screw using a T-wrench.

Only 1-2mm of the screw tip should penetrate the second cortex to avoid the risk of soft tissue damage.






WARNING: Do not excessively penetrate the second cortex with any type of screws to avoid soft tissue damage.




PRECAUTION: Any attempt to back out a conical screw once it has been inserted may cause it to become loose.

- Insert the threaded screw guide fully into the screw template; locate it in the center of the bone axis, and repeat the above procedure for the distal metacarpal screw.

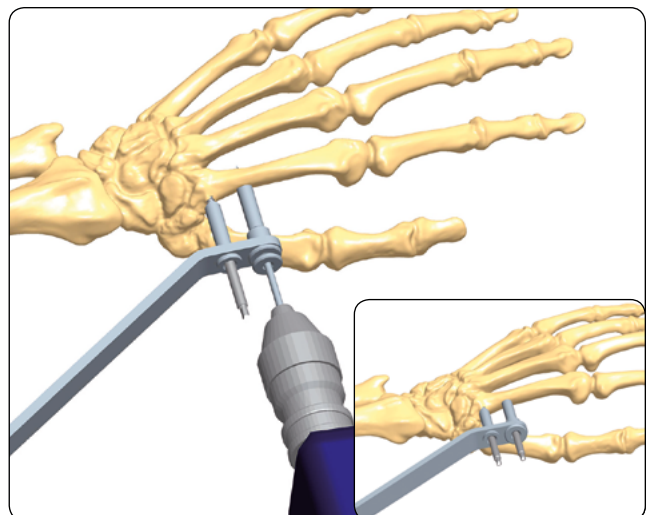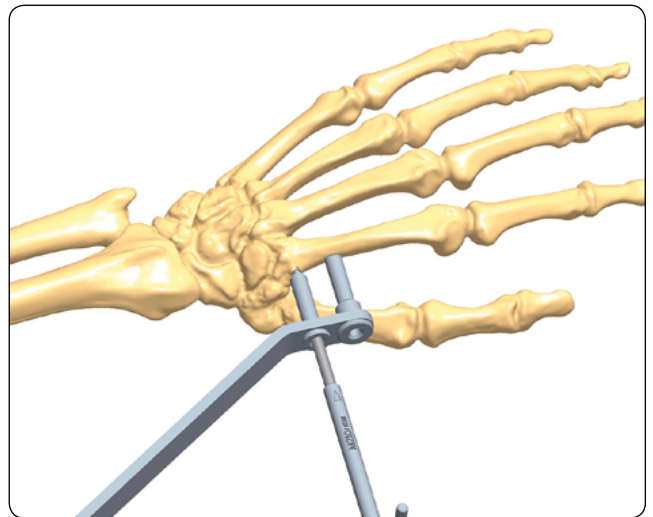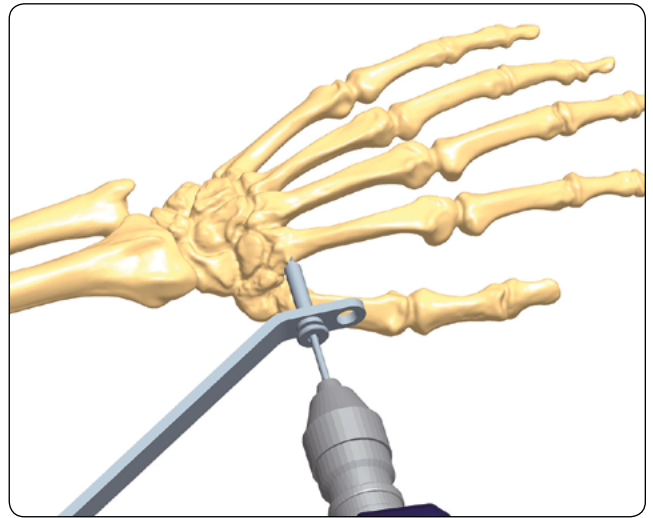


INSERTION OF RADIAL SCREWS

- Insert the radial screws using an open approach. Mount the fixator temporarily on the distal screws and mark the position of the proximal screws on the skin. Ensure that the fixator body is partially open, to allow for intra-operative distraction or compression of the fracture. Make a 3 cm incision, and expose the bone by blunt dissection to avoid injury to the superficial branch of the radial nerve. Insert two 70/20 radial screws using the same technique as above.

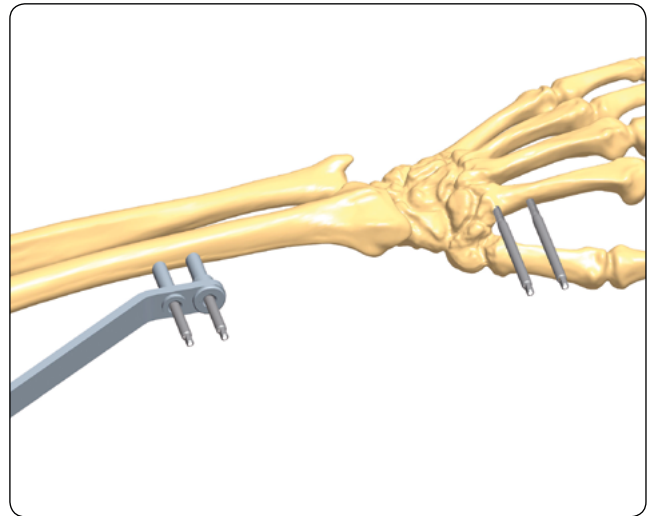

- Place the fixator over the screws with cams and central body locking nut loosened, ensuring that the central body locking nut does not lie over the fracture site since it is not radiolucent. Tighten the clamp cover locking screws with the 4mm Allen Wrench. Reduce the fracture and tighten the cams and central body locking nut with the 4mm Allen Wrench.

The cams only tighten clockwise, as indicated by the arrow on the fixator clamp.

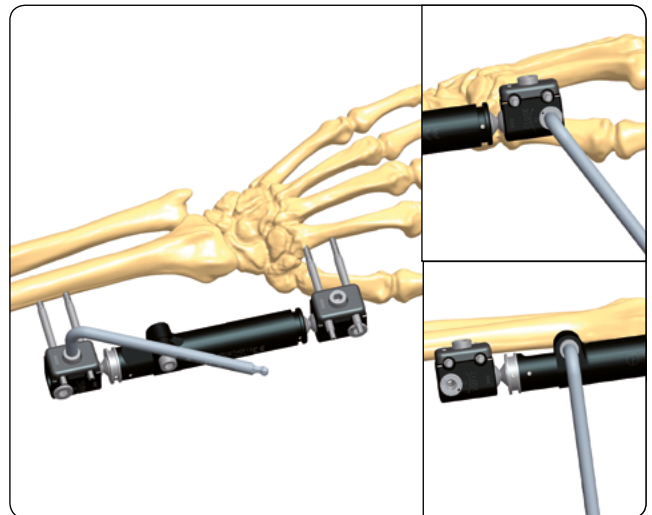


- When using the Compression-Distraction Unit, locate the clamps in the slots on the body of the fixator and tighten the two set screws. To achieve controlled compression or distraction, loosen the central body locking nut and turn the central element of the compression-distraction unit. Once this has been achieved, tighten the central body locking nut and remove the Compression-Distraction Unit.

Check the amount of wrist joint distraction by X-ray before the patient leaves the operating room to ensure that the joint is not left in over-distraction.



PRECAUTION: During and after insertion of the implants, ensure their correct positioning under image intensification.

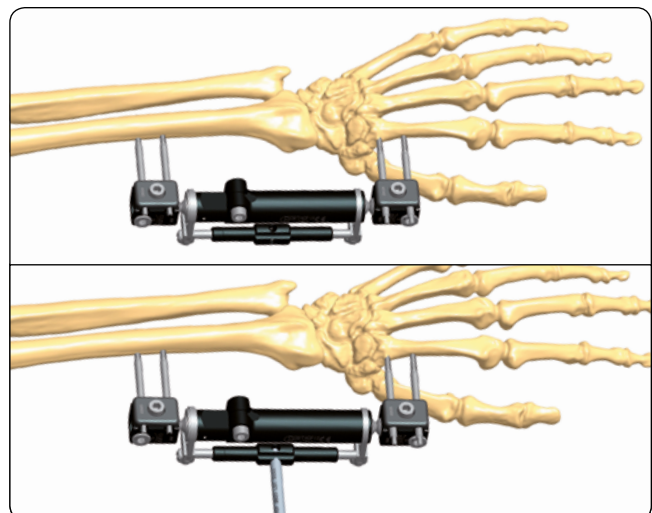


Electronic Instructions for use available at the website
<http://ifu.orthofix.it>

Electronic Instructions for use - Minimum requirements for consultation:

- Internet connection (56 Kbit/s)
- Device capable to visualize PDF (ISO/IEC 32000-1) files
- Disk space: 50 Mbytes

Free paper copy can be requested from customer service
(delivery within 7 days):
tel +39 045 6719301, fax +39 045 6719370,
e-mail: customerservice@orthofix.it

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Distributed by:



Manufactured by:
ORTHOFIX Srl
Via Delle Nazioni 9, 37012 Bussolengo (Verona), Italy
Telephone +39 045 6719000, Fax +39 045 6719380



Caution: Federal law (USA) restricts this device to sale by or on the order of a physician. Proper surgical procedure is the responsibility of the medical professional. Operative techniques are furnished as an informative guideline. Each surgeon must evaluate the appropriateness of a technique based on his or her personal medical credentials and experience. Please refer to the "Instructions for Use" supplied with the product for specific information on indications for use, contraindications, warnings, precautions, adverse reactions and sterilization.