EN	ORTHOFIX® LRS DYNA BLOCK 2	ZH	ORTHOFIX® LRS DYNA BLOCK 28
IT	ORTHOFIX® LRS DYNA BLOCK 4	CS	ORTHOFIX® LRS DYNA BLOCK 30
FR	ORTHOFIX® LRS DYNA BLOCK 6	PL	LRS DYNA BLOCK ORTHOFIX® 32
DE	ORTHOFIX® LRS DYNA BLOCK 8	SL	ORTHOFIX® LRS DYNA BLOCK 34
ES	ORTHOFIX® LRS DYNA BLOCK 10	KO	ORTHOFIX® LRS DYNA BLOCK 36
DA	ORTHOFIX® LRS DYNA BLOCK 12	AR	ORTHOFIX® LRS DYNA BLOCK 38
FI	ORTHOFIX® LRS DYNA BLOCK 14	BG	ORTHOFIX® LRS DYNA BLOCK 40
NO	ORTHOFIX® LRS DYNA BLOCK 16	LT	ORTHOFIX® LRS DYNA BLOCK 42
NL	ORTHOFIX® LRS DYNA BLOCK 18	MS	ORTHOFIX® LRS DYNA BLOCK 44
PT	ORTHOFIX® LRS DYNA BLOCK 20	RO	ORTHOFIX® LRS DYNA BLOCK 46
SV	ORTHOFIX® LRS DYNA BLOCK 22	TR	ORTHOFIX® LRS DYNA BLOCK 48
EL	ORTHOFIX® LRS DYNA BLOCK 24	HR	ORTHOFIX® LRS DYNA BLOCK 50
JA	ORTHOFIX® LRS DYNA BLOCK 26	LV	ORTHOFIX® LRS DYNA BLOCK 52

ORTHOFIX Srl
Via Delle Nazioni 9 - 37012 Bussolengo (Verona) - Italy
Tel. 0039 (0) 45 6719000 - Fax 0039 (0) 45 6719380

PQDYB B 05/21 (0418427)

EN

Instructions For Use (IFU) are subject to change; the most current version of each IFU is always available online

Important information - please read prior to use

ORTHOFIX® LRS DYNA BLOCK

  **Important information - please read prior to use**

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Tel. 0039 (0) 45 6719000 - Fax 0039 (0) 45 6719380

LRS Dyna Block (SS53537, SS55537)

The Dyna Block is available both in adult (SS53537) and paediatric (SS55537) model. The Adult Dyna Block is used in association with a LRS ADV rail, while the Paediatric Dyna Block with a Paediatric LRS rail. Either Dyna Blocks are used with a Sheffield or a Truelok Ring.

Indication for use:

Protection of an unstable knee during lengthening according to the indications and contraindications listed in the PQREC.

Warnings

- Position the Hinge Pin in the Center of rotation of the knee. Wrong positioning could limit the R.O.M. of the knee.
- Do not insert more than one washer on the Hinge Pin between the Sheffield Clamp and the Dyna Block, otherwise the stiffness of the construct could decrease significantly.

Precautions

- The Hinge Pin must always be inserted into one of the Dyna Block holes closest to the distal diaphyseal bone screws.
- Ensure that the Sheffield Clamp is placed as close as possible to the Dyna Block.
- Once all bone screws have been inserted, check that flexion-extension of the knee is not impeded. This will confirm correct positioning of the hinge. If necessary, replace the K-wire and Hinge Pin.
- The SS53537 and SS55537 have not been tested in traumatic and severely unstable knee conditions. These could require an increased stiffness to ensure congruent healing and a more anatomical rotation to avoid worsening of joint conditions.
- An additional stabilizing rod should be applied anteriorly, between the anterior face of the Dyna Block and the Ring, to span the hinge and protect from a knee flexion contracture when knee movement is not controlled (e.g. patient is asleep). (Use only the central threaded hole present in the Dyna Block to fix the stabilizing rod).
- Lengthening of the femur along mechanical axis could generate anatomical axis mal-alignment. Adequate preoperative planning of corrective procedures should be performed to avoid this condition.

Symbol	Description	
	Medical Device	
 	Consult instructions for use or consult electronic instructions for use	Caution: Consult instructions for use for important cautionary information
	Single Use. Do not re-use	Orthofix note: discard appropriately after the use (treatment) on patient
	Non-sterile	
 	Catalogue number	Batch code
	Use-by date (year-month-day)	
	CE marking in conformity to applicable European Medical Device Directives/Regulations	
 	Date of manufacture	Manufacturer
	Do not use if package is damaged and consult instructions for use	
Rx Only	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician	
	Unique Device Identifier	

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Orthofix Srl