JA	EL	SV	PT	NL	NO	FI	DA	ES	DE	FR	IT	EN	ORTHOFIX® TRUELOK - FOREFOOT ARCH 2	ORTHOFIX® TRUELOK - FOREFOOT ARCH 4	ORTHOFIX® TRUELOK - FOREFOOT ARCH 6	ORTHOFIX® TRUELOK - FOREFOOT ARCH 8	ORTHOFIX® TRUELOK - FOREFOOT ARCH 10	ORTHOFIX® TRUELOK - FOREFOOT ARCH 12	ORTHOFIX® TRUELOK - FOREFOOT ARCH 14	ORTHOFIX® TRUELOK - FOREFOOT ARCH 16	ORTHOFIX® TRUELOK - FOREFOOT ARCH 18	ORTHOFIX® TRUELOK - FOREFOOT ARCH 20	ORTHOFIX® TRUELOK - FOREFOOT ARCH 22	ORTHOFIX® TRUELOK - FOREFOOT ARCH (Προπορευόμενου τόξου) 24	ORTHOFIX® TRUELOK - FOREFOOT ARCH 26	ORTHOFIX® TRUELOK - FOREFOOT ARCH 28	ORTHOFIX® TRUELOK - FOREFOOT ARCH 30	ORTHOFIX® TRUELOK - FOREFOOT ARCH 32	ORTHOFIX® TRUELOK - FOREFOOT ARCH 34	ORTHOFIX® TRUELOK - FOREFOOT ARCH 36	ORTHOFIX® TRUELOK - FOREFOOT ARCH 38	ORTHOFIX® TRUELOK - FOREFOOT ARCH 40	ORTHOFIX® TRUELOK - FOREFOOT ARCH 42	ORTHOFIX® TRUELOK - FOREFOOT ARCH 44	ORTHOFIX® TRUELOK - FOREFOOT ARCH 46	ORTHOFIX® TRUELOK - Ön Ayak Kemerı 48	ORTHOFIX® TRUELOK - FOREFOOT ARCH 50	ORTHOFIX® TRUELOK - FOREFOOT ARCH 52
----	----	----	----	----	----	----	----	----	----	----	----	----	--	--	--	--	---	---	---	---	---	---	---	--	---	---	---	---	---	---	---	---	---	---	---	--	---	---

ORTHOFIX Srl  
Via Delle Nazioni 9 - 37012 Bussolengo (Verona) - Italy  
Tel. 0039 (0) 45 6719000 - Fax 0039 (0) 45 6719380

PQ FRA B 03/21 (0421217)

Instructions For Use are subject to change; the most current version of each Instruction For Use is always available online.


**Important information - please read prior to use**  
**ORTHOFIX® TrueLok - Forefoot Arch**

56-14140 - TRUELOK PLUS FOREFOOT REDUCTION ARCH 140MM

56-14160 - TRUELOK PLUS FOREFOOT REDUCTION ARCH 160MM

**ORTHOFIX SRL, via Delle Nazioni 9, 37012 Bussolengo (VR) Italy**  
Tel. 0039 (0) 45 6719000 - Fax 0039 (0) 45 6719380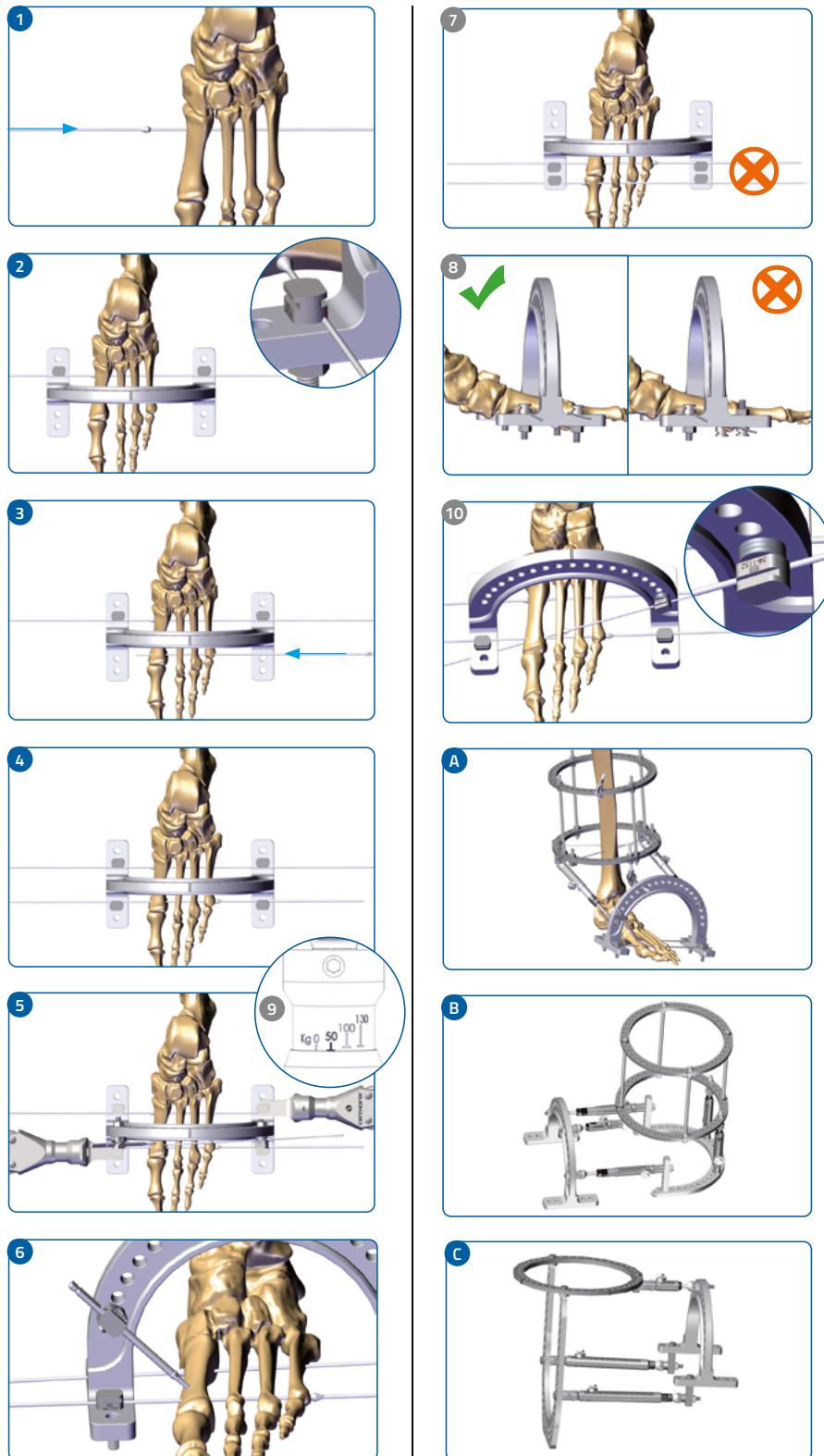

- The TrueLok Forefoot Arch is compatible with TrueLok and TL-HEX frame.
- Patient should be warned about hazards related to weight bearing, including failure of the frame.
- For important medical information and maintenance consult Instructions for Use PQ TLK.

#### **TrueLok Forefoot Arch application (Refer to pictures)**

- Insert the first wire with olive across the foot at the level of the proximal portion of the metatarsus.
- Link the wire to the stands of the Forefoot Arch.
- Using the Forefoot Arch as a template put the second wire with the olive on the opposite side of the foot.
- Link the second wire to the stands of the Forefoot Arch.
- Tension the wires as appropriate according to bone stock quality.
- If needed, put additional wires or half pin across the foot.

#### **TrueLok Forefoot Arch application - Precautions (Refer to pictures)**

- At least one wire must be inserted on either side of the Forefoot Arch.
- Wires must all be either positioned above or below the stands of the Forefoot Arch. Variable wire positioning above and below the stands must be avoided.
- 50kgs for wire tensioning is recommended, paying attention the olive on the wire will not go into the bone.
- When attaching additional wires to the Forefoot Arch, washers must be used to avoid impingement on the arch lip and bending of the wire.

#### **TrueLok Forefoot arc connection (Refer to pictures)**

- Optionally, the Forefoot Arch can be connected to TrueLok or TL-HEX frame.
- Alternative option to connect the dorsal aspect of the Forefoot Arch to a frame using struts.
- Alternative option to connect the dorsal aspect of the Forefoot Arch to a heel ring using struts.

Symbol	Description	
<b>MD</b>	Medical Device	
 	Consult instructions for use or consult electronic instructions for use	Caution: Consult instructions for use for important cautionary information
	Single Use. Do not re-use	Orthofix note: discard appropriately after the use (treatment) on patient
<b>STERILE R</b>	Sterile. Sterilised by irradiation	
	Non-sterile	
	Double sterile barrier system	
<b>REF</b> <b>LOT</b>	Catalogue number	Batch code
	Use-by date (year-month-day)	
<b>CE</b> <b>CE</b> 0123	CE marking in conformity to applicable European Medical Device Directives/Regulations	
 	Date of manufacture	Manufacture
	Do not use if package is damaged and consult instructions for use	
<b>Rx Only</b>	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician	

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Orthofix Srl