EN	ORTHOFIX® TrueLok RockerRail 2	ZH	ORTHOFIX® TrueLok RockerRail 67
IT	ORTHOFIX® TrueLok RockerRail 7	CS	ORTHOFIX® TrueLok RockerRail 72
FR	ORTHOFIX® TrueLok RockerRail 12	PL	ORTHOFIX® TrueLok RockerRail 77
DE	ORTHOFIX® TrueLok RockerRail 17	SL	ORTHOFIX® TrueLok RockerRail 82
ES	ORTHOFIX® TrueLok RockerRail 22	KO	ORTHOFIX® TrueLok RockerRail 87
DA	ORTHOFIX® TrueLok RockerRail 27	AR	ORTHOFIX® TrueLok RockerRail 92
FI	ORTHOFIX® TrueLok RockerRail 32	BG	ORTHOFIX® TrueLok RockerRail 97
NO	ORTHOFIX® TrueLok RockerRail 37	LT	ORTHOFIX® TrueLok RockerRail 102
NL	ORTHOFIX® TrueLok RockerRail 42	MS	ORTHOFIX® TrueLok RockerRail 107
PT	ORTHOFIX® TrueLok RockerRail 47	RO	ORTHOFIX® TrueLok RockerRail 112
SV	ORTHOFIX® TrueLok RockerRail 52	TR	ORTHOFIX® TrueLok RockerRail 117
EL	ORTHOFIX® TrueLok RockerRail 57	HR	ORTHOFIX® TrueLok RockerRail 122
JA	ORTHOFIX® TrueLok RockerRail 62	LV	ORTHOFIX® TrueLok RockerRail 127

EN

Instructions For Use are subject to change; the most current version of each Instruction For Use is always available online.

CE

Important information - Please read prior to use
ORTHOFIX® TrueLok RockerRail

Orthofix Srl - Via delle Nazioni 9 - 37012 Bussolengo (VR) Italy - Telephone +39 045 671 9000 - Fax +39 045 671 9380

- A. The TrueLok RockerRail is compatible with TrueLok and TL-HEX Foot Plates.
- B. Patient should be warned about hazard related to slippage.
- C. For important medical information and maintenance consult Instructions for Use PQ TLK.
- D. Steam sterilization not allowed.
- E. Remove the caps from both Rocker Rail ends before application.

TrueLok RockerRail Application (Refer to pictures)

- 1. Remove the outer locking nut from the fixation rods on the rails.
- 2. Loosen the adjustable sliding fixation.
- 3. Place the TrueLok RockerRail with the FRONT marking anteriorly onto the Foot Plate by inserting its fixation rods into the appropriate holes.
- 4. Place locking nuts back onto the fixation rods of the rails.
- 5. Use two 10mm wrenches to lock the rails onto the Foot Plate firmly.
- 6. The rails should be adjusted as needed for ideal placement. Lock the adjustable sliding fixation.

TrueLok RockerRail Height Adjustment

- 7. In the case of needing longer threaded rods, please remove the provided rods with the 1/8" Allen wrench.
- 8. Exchange the rods with the threaded rods of the desired length.
- 9. If threaded rods longer than 60mm are necessary, strengthen the construct with the Rocker Rail Extension Kits. Do not use threaded rods longer than 115mm. In case a greater extension is needed, consider adding an additional footplate.
- 10. The larger side of the Extender must be in direct contact with the footplate.
- 11. To adjust the distance between Footplate and Rocker Rail insert nuts and/or extended nuts between the Extender and the Rocker Rail. Do not allow any portion of bare rod.

TrueLok RockerRail Final Check (Warnings & Precautions)

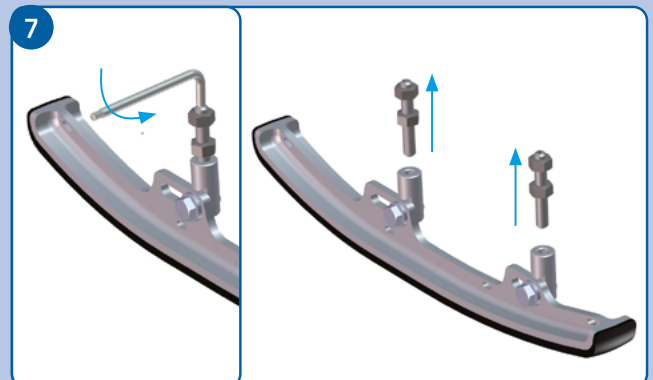
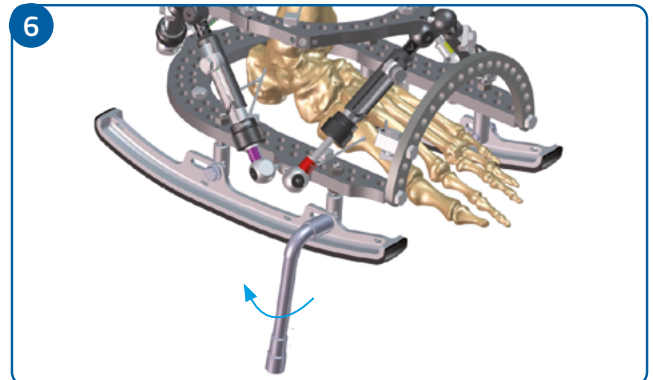
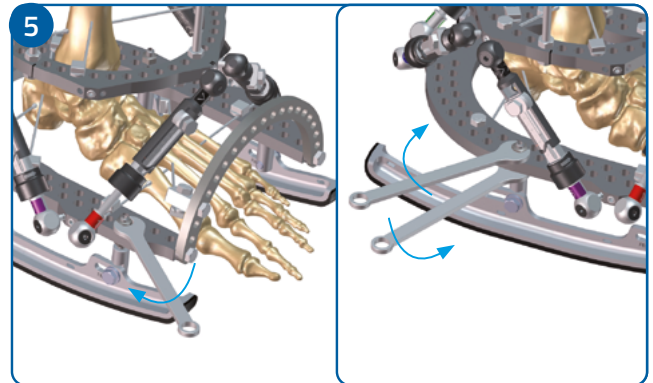
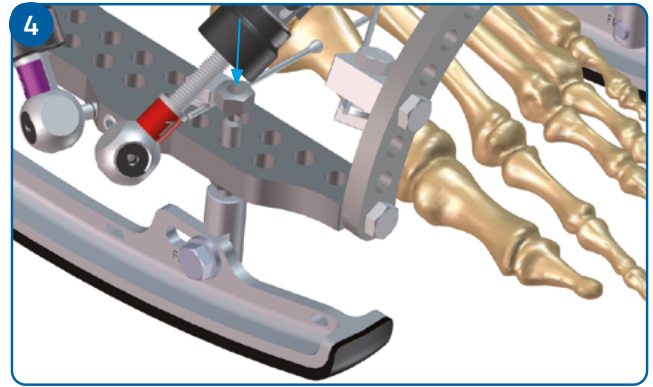
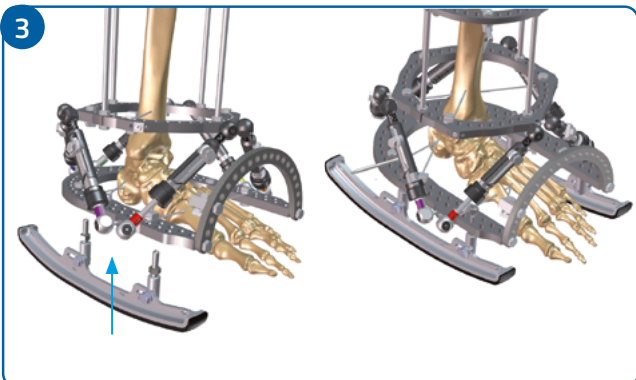
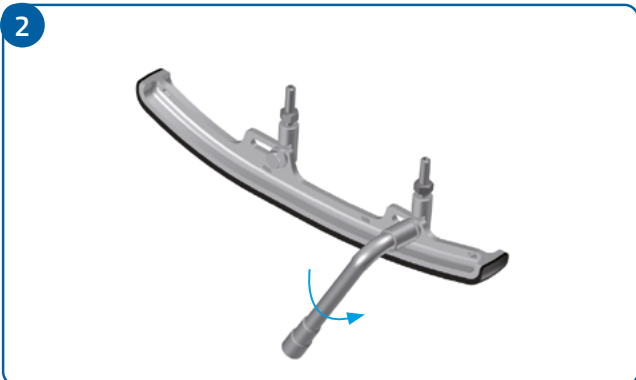
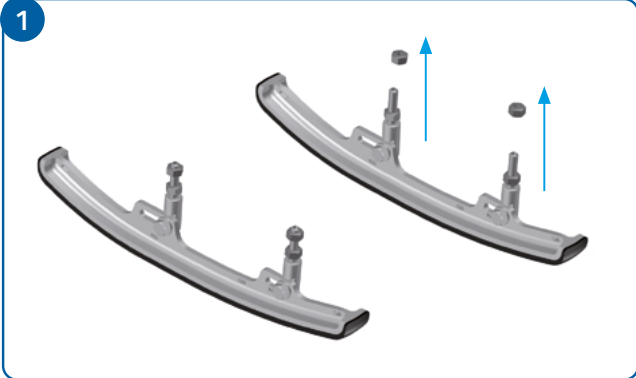
- 12. Make sure that the two rails are placed onto the Foot Plate to provide a balanced platform to the foot, keeping it in a neutral position.
- 13. The rails can be placed unevenly relative to each other but, they should keep the foot in a neutral position.
- 14. ⚠ The rails should never be placed in a way to pronate or supinate the foot in static position.
- 15. The front and the rear of the rails should always be in line with each other.
- 16. ⚠ The rails should never be placed offset to each other.
- 17. Do not put nuts between the Footplate and the Extender.
- 18. Use maximum one Extension Tube per threaded rod.

NOTE FOR USE


Do not use Orthofix products in conjunction with those of other manufacturers, unless otherwise specified, as the combination is not covered by the necessary validation.

TrueLok RockerRail


56-24000


8

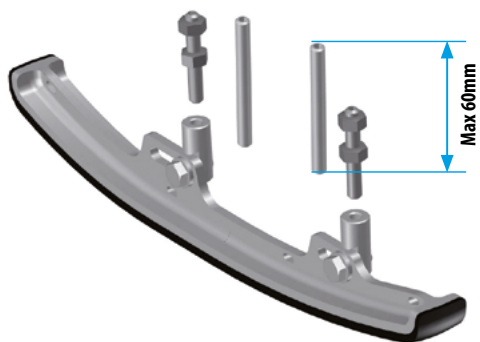
9

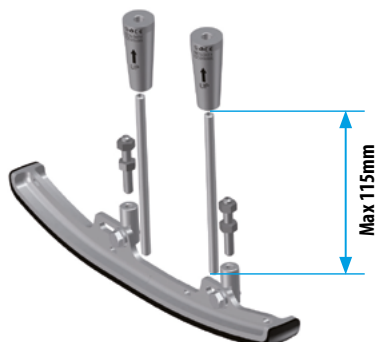
10

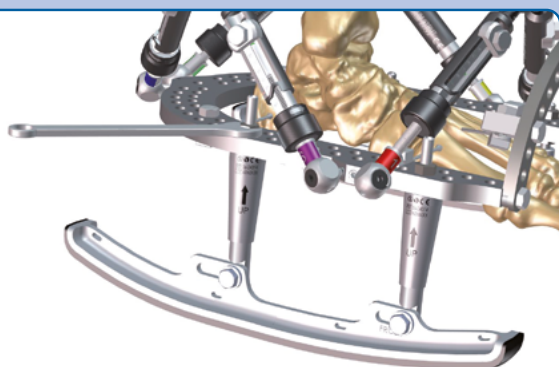
11

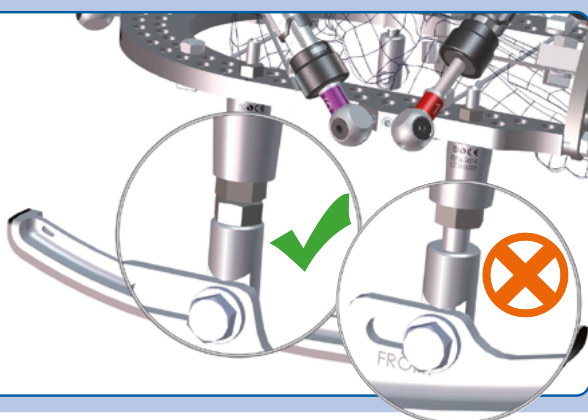
12

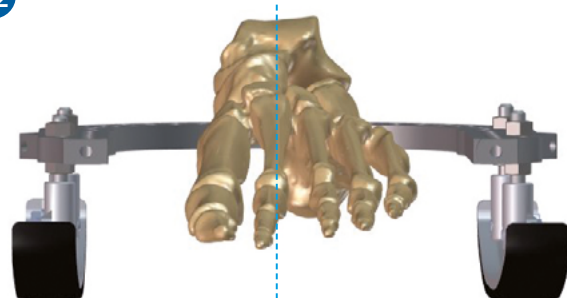13

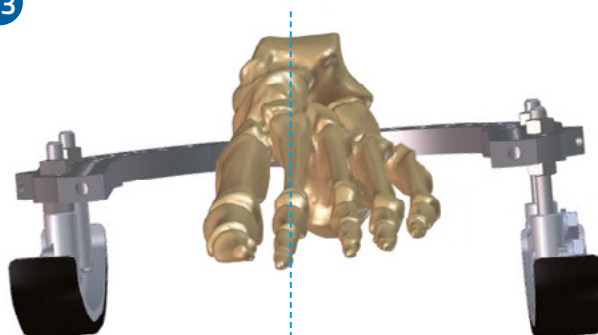
14

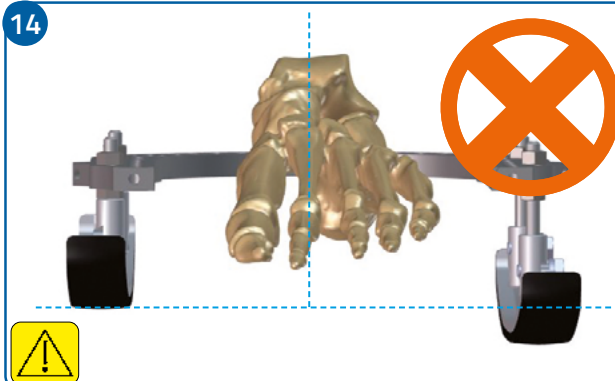15

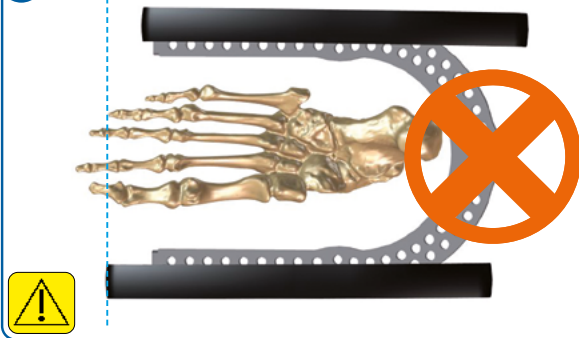16

17

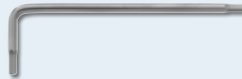
56-24014

56-24000

Instruments

	Part#	Qty.
	52-1020	1
	54-1154	2
	54-2226	1

Symbol	Description	
	Medical Device	
 	Consult instructions for use or consult electronic instructions for use	Caution: Consult instructions for use for important cautionary information
	Single Use. Do not re-use	Orthofix note: discard appropriately after the use (treatment) on patient
	Sterile. Sterilised by irradiation	
	Non-sterile	
	Double sterile barrier system	
 	Catalogue number	Batch code
	Use-by date (year-month-day)	
  0123	CE marking in conformity to applicable European Medical Device Directives/Regulations	
 	Date of manufacture	Manufacture
	Do not use if package is damaged and consult instructions for use	
Rx Only	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician	

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